

## Kirby Lester® KL15e Easy-Start Guide

### Set-up:

You will need no tools and only 10 minutes to begin using your KL15e. It's that easy!

1. Remove the unit from the shipping box and place the unit on the pharmacy counter. **NOTE:** Save the shipping box and packing materials in case the unit needs to be serviced in the future.
2. Assemble the two funnel pieces and place it into the square collar on the top of the unit.
3. Slide the counting tray completely into the bottom front.
4. Connect the power cord to the right side of the unit and plug the other end into a standard 110-volt outlet. Turn the power switch to "On".



### Operation:

1. Once three **red** zeros appear on the main screen, the unit is ready.
2. Pour tablets/capsules as close to the **outer edge** of the funnel as possible so they gently tumble into the hole. **Do not pour tablets directly into the funnel hole.** This will cause counting errors.
3. As you approach your target count, slow your pouring speed and stop at your target count (displayed in **red** on-screen).
4. Select a vial according to the Container Size indicator (**green**, in drams).
5. Pour the tablets from the tray into the vial. Return the tray, inserting completely.
6. Press Zero and three **red** Zeros will appear on-screen. You are ready to start the next order.



### Error Message – Pouring Too Quickly:

The Kirby Lester unit will display an error message when you are pouring too quickly.

- The **Slow !** indicator will illuminate in **orange**. Slow down immediately.
- If you continue to pour too quickly, the Quantity display will read **O-S (Over Speed)**.
- Stop, empty pills from the tray, press Zero and repeat the count.

## Special Mode:

Use the “**Special Mode**” setting to count unique/odd-shaped tablets like Valium (hole) or Maxzide (odd shape).

- To engage Special Mode, press the Zero button – **twice** – in rapid succession.
- A light will indicate that Special Mode is on. Process your order.
- To return to Normal Mode, press the Zero button – **once** - and the light will turn off.


**Note:** Kirby Lester units **will not** accurately count **clear** capsules.

## Cleaning/Maintenance:

Clean the following components at least once daily. Clean more frequently if your pharmacy dispenses many dust-generating medications or if you experience counting errors.

**Clean with 70% isopropyl alcohol or water and lint-free cloth:**

- Funnel – remove and clean outside and inside thoroughly with alcohol.
- Four Glass Detector Windows – inside the square funnel hole.  
**Most counting errors are caused by pill dust accumulating on the edges and corners of the windows.**  
**CLEAN THOROUGHLY** with alcohol. Wipe each side several times to remove all dust and residue. Wipe dry with clean cloth to remove alcohol streaks.
- Counting Tray – carefully remove the clear plastic lid. Clean all parts thoroughly with a damp cloth (not alcohol). Wipe dry. Replace the plastic lid. Turn the tray over, and wipe the silver sensor strip with a damp cloth (not alcohol).
- Tray Sensor Window – this sensor touches the tray’s silver sensor strip, telling the unit that the tray is properly in place. It is located on the bottom of the unit in back of the counting tray bay. With the counting tray removed, tip the unit at a 45-degree angle to access the tray sensor window. Clean with a damp cloth (not alcohol).

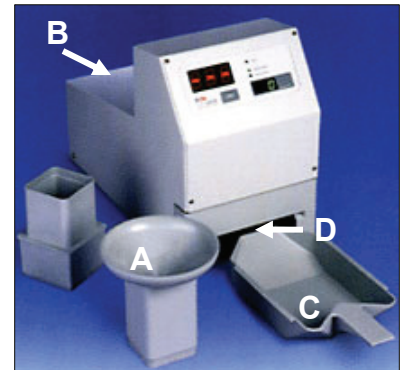

## Kirby Lester Operating Manual and Customer Service:

Please refer to the **Kirby Lester Operating Manual** for detailed set-up, operation and maintenance instructions. If you still need help, contact Kirby Lester Customer Service: **800.243.2465** or **service@kirbylester.com**.